



iTouch/iPhone Install and Launch and Demo Steps

The following steps should be executed to install the AirStrip OB client onto the iTouch or iPhone:



- 1) Go to the App Store and search for AirStrip OB, select install. This is a free app.

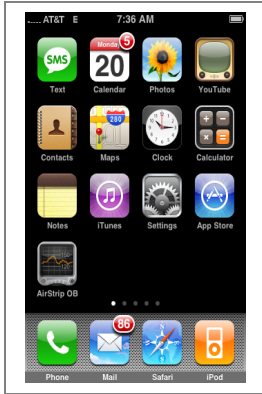

- 2) Once the application is installed, complete the registration.

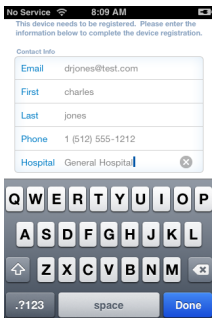


- 3) Select the Demo/Training Hospital, no log in credentials required. Just hit Go

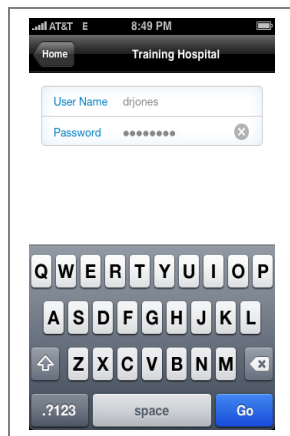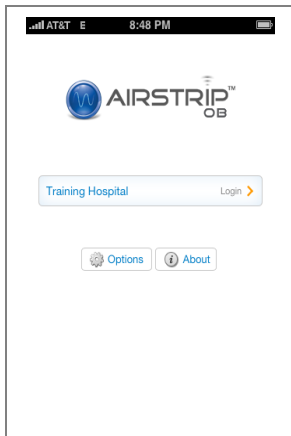


4) View census (all laboring patients)

Note – colored dots of the left quickly show who is furthest along in her labor. Additional summary information on each patient is available at a glance.

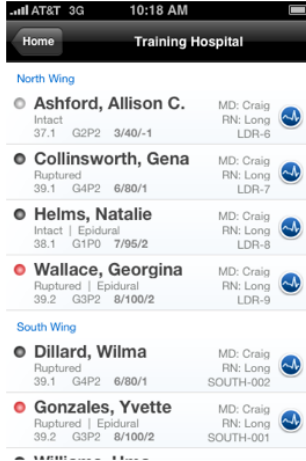


5) Touch “Georgina Wallace” to further access more clinical data

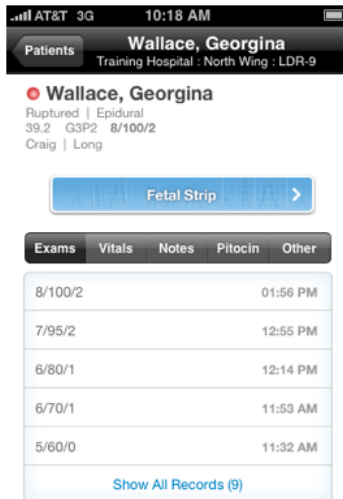


See cervical exam history, for example.

a. Touch Vitals – drill down into history and trends

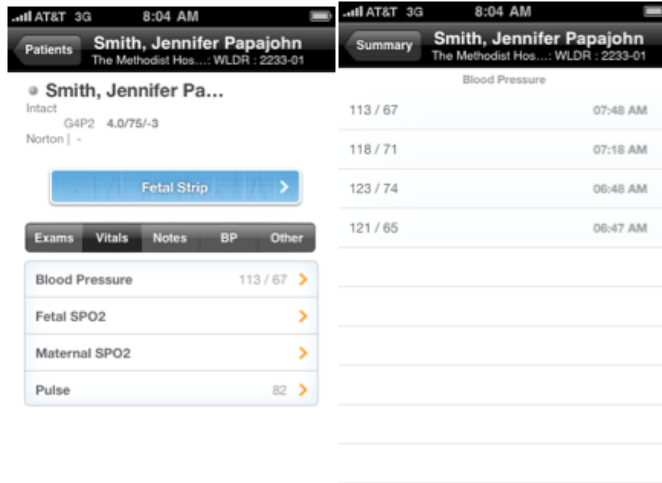

b. Touch Nurses Notes – can be viewed in summary or in sequential detail

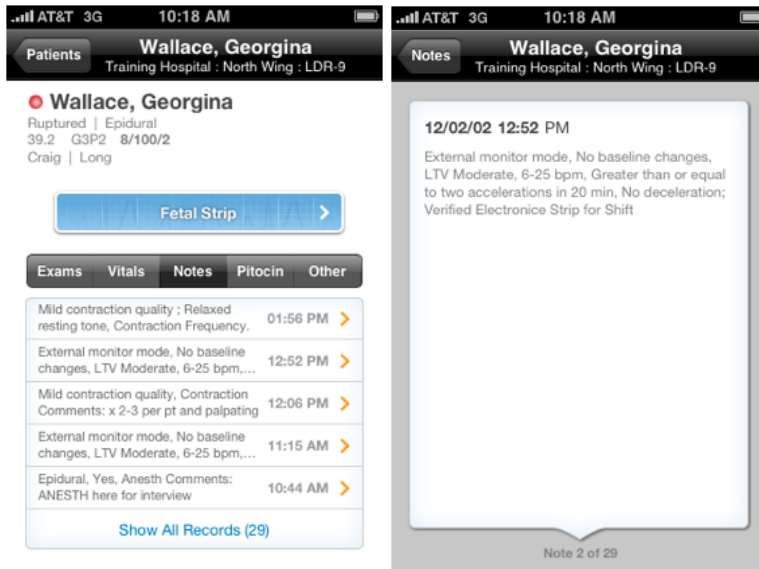


c. Touch Pitocin (meds administration history)

d. Other (this tab can be customized to your hospital needs and is a function of all the additional clinical information you may capture on your perinatal information system today)



- 6) Touch blue-colored Fetal Strip to view real-time waveform data and historic maternal and fetal waveform data

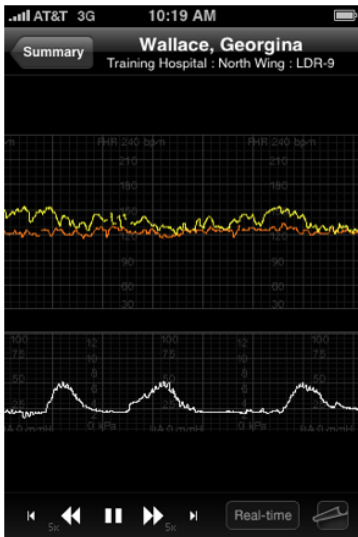

- 7) Rotate to landscape and swipe finger from left to right to view historic waveform data

